


Which Size Boot Do I Need?

Get the right size the first time by following these simple steps


- 1** Find a sturdy piece of paper or cardboard and place your horse's hoof squarely on it.

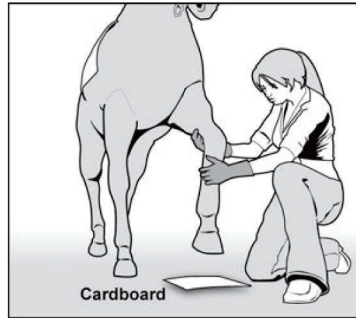

- 2** Use a short dark pencil (a golf pencil, for example). It must be able to run under the ankle at the back.


Trace around the foot holding the pencil as shown. You will need a tight trace with the pencil vertical. Trace around the heel bulbs.

Mark end of ground contact area on both sides of the hoof. (Shod or unshod) Ground contact area must fit within gel orthotic surface area.

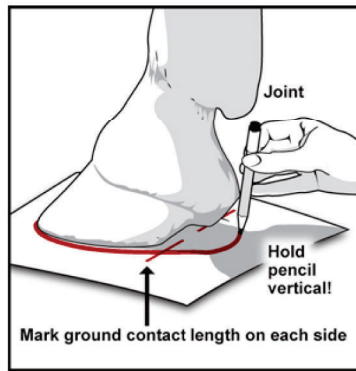

- 3** Now measure your trace at:

- The widest point, as shown
- The longest ground contact point, as shown.
- The longest point including bulbs, as shown.

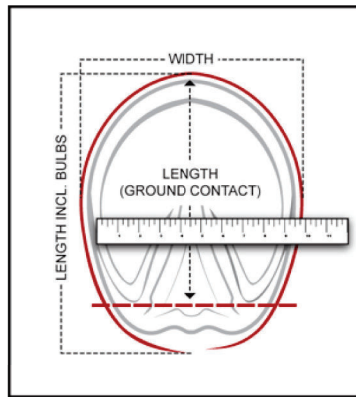

- 4** Consult the size chart to see which gel orthotic / boot size to order.

Note: Some "underslung hoofs" may need next size up

SIZE CHART

WIDTH	LENGTH	LENGTH INC. BULBS	SIZE	PART #
7 1/2 in. 196 mm	7 3/4 in. 197 mm	8 1/2 in. 217 mm	1	SR1001
6 1/2 in. 165 mm	7 1/8 in. 181 mm	8 in. 203 mm	2	SR1002
6 1/4 in. 159 mm	6 3/8 in. 162 mm	7 1/4 in. 185 mm	3	SR1003
5 3/4 in. 143 mm	6 in. 156 mm	6 3/4 in. 172 mm	4	SR1004
5 5/8 in. 153 mm	6 1/8 in. 155 mm	7 in. 178 mm	5L	SR1005 LONG
5 3/4 in. 146 mm	5 3/4 in. 146 mm	6 1/2 in. 165 mm	5	SR1005
5 1/4 in. 134 mm	5 3/4 in. 146 mm	6 3/4 in. 172 mm	6L	SR1006 LONG
5 1/4 in. 134 mm	5 1/4 in. 134 mm	6 in. 153 mm	6	SR1006
4 3/4 in. 121 mm	5 1/4 in. 134 mm	6 in. 153 mm	7L	SR1007 LONG
4 3/8 in. 111 mm	4 3/4 in. 121 mm	5 1/2 in. 140 mm	7	SR1007
3 3/4 in. 95 mm	4 in. 102 mm	4 3/4 in. 122 mm	8	SR1008
3 1/8 in. 80 mm	3 5/8 in. 93 mm	4 1/2 in. 115 mm	9	SR1009
2 3/4 in. 70 mm	2 5/8 in. 67 mm	3 1/4 in. 83 mm	10	SR1010

**Yellow highlighted products indicate experimental products that are only available in special circumstances. Please call to order these items.

[Click here to download these instructions, including the chart in a handy, printable 8 1/2"x11" PDF format. Take it with you to the stall!](#)